
BE IT KNOWN that I, Sunita Kumeri of 18-24 Stoke Road, Slough, Berkshire United Kingdom a duly authorised Notary Public

CERTIFY ONLY that the signature set and subscribed at the foot of the document annexed hereto is genuine having been subscribed thereto by Emma Jane Hughes whose identity I the Notary attest, who was by a Power of Attorney dated the 26th March 2012 appointed as Attorney by Reckitt Benckiser Healthcare (UK) Limited (a private limited company incorporated in England and Wales on the 22nd December 1931 with company registration number 00261312) and is thereby authorised to sign the annexed document on its behalf.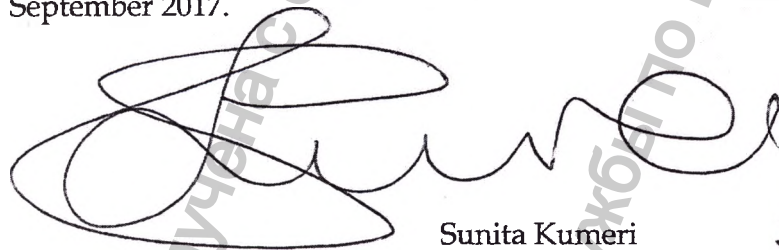

SIGNED and sealed at 18-24 Stoke Road, Slough, Berkshire aforesaid on 5th September 2017.

Sunita Kumeri
Notary Public
England & Wales

Protocol No. 2/17


APOSTILLE (Convention de La Haye du 5 octobre 1961)	
1. Country: Pays / Pais:	United Kingdom of Great Britain and Northern Ireland
This public document Le présent acte public / El presente documento público	
2. Has been signed by a été signé par ha sido firmado por	Sunita Kumeri
3. Acting in the capacity of agissant en qualité de quien actúa en calidad de	Notary Public
4. Bears the seal / stamp of est revêtu du sceau / timbre de y está revestido del sello / timbre de	The Said Notary Public
Certified Attesté / Certificado	
5. at à / en	London
6. the le / el día	06 September 2017
7. by par / por	Her Majesty's Principal Secretary of State for Foreign and Commonwealth Affairs
8. Number sous no / bajo el numero	APO-522094
9. Seal / stamp Sceau / timbre Sello / timbre	
10. Signature Signature Firma	D. O'Sullivan 

This Apostille is not to be used in the UK and only confirms the authenticity of the signature, seal or stamp on the attached UK public document. It does not confirm the authenticity of the underlying document. Apostilles attached to documents that have been photocopied and certified in the UK confirm the signature of the UK official who conducted the certification only. It does not authenticate either the signature on the original document or the contents of the original document in any way.

If this document is to be used in a country not party to the Hague Convention of the 5th of October 1961, it should be presented to the consular section of the mission representing that country

To verify this apostille go to www.verifyapostille.service.gov.uk

Approved by

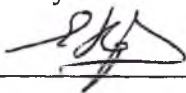Date: 21/05/17

(Signature)

Name of Responsible Person *EMMA HUOHES*

Post *GRA- SENIOR MANAGER*

Stamp

Operational documentation:

Durex® Play lubricants

Manufactured by:

Reckitt Benckiser Healthcare (UK) Ltd., UK

**Reckitt Benckiser
Healthcare (UK) Limited**
Dansom Lane, Hull, HU8 7DS

Registered Office: 103-105 Bath
Road, Slough, SL1 3UH
Company No. 261312

Operational documentation

Durex® Play lubricants in variations

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Instruction for use ¹

medical device "Durex® Play lubricants in variations", legal manufacturer
Reckitt Benckiser Healthcare (UK) Ltd., United Kingdom

DEVICE NAME

Play lubricants in variations"
Durex® Play Feel lube;
Durex® Play Tingle lube;
Durex® Play Heat lube;
Durex® Play Sweet Strawberry lube;
Durex® Play Very Cherry lube.

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Device description

Durex® Play Sweet Strawberry, Durex® Play Very Cherry and Durex® Play Tingle are flavored lubricants

Durex® Play Feel and Durex® Play Heat are flavorless lubricants.

The flavors in Durex® Play Sweet Strawberry, Durex® Play Very Cherry are achieved by the following ingredients, such as Strawberry flavoring agent, Cherry flavoring agent, respectively. Durex® Play Tingle contains multi-sensate, a specific ingredient which cools and tingles at the same time. It also gives Durex® Play Tingle a specific odour but no taste.

Durex® Play Heat has a specific warming effect, which is achieved by hygroscopic effect in contact with the skin due to the high concentration of glycerol.

¹ The corresponding information included on leaflet or on the bottle of medical device

Composition

Product name	Chemical compound
Play Feel lube	Purified water
	Propylene glycol
	Hydroxyethylcellulose
	Benzoic acid
	Sodium hydroxide
Durex® Play Heat lube	Glycerol
	Purified water
	Propylene glycol
	Hydroxyethylcellulose
	Benzoic acid
	Saccharin sodium
	Sodium hydroxide
Durex® Play Strawberry lube	Purified water
	Hydroxyethylcellulose
	Propylene glycol
	Benzoic acid
	Sodium hydroxide
	Saccharin sodium
	Glycerol
	Strawberry flavoring agent
Durex® Play Tingle lube	Purified water
	Propylene glycol
	Hydroxyethylcellulose
	Benzoic acid
	Sodium hydroxide
	Multi Sensate
	Saccharin sodium
Durex® Play Cherry lube	Glycerol
	Purified water
	Propylene glycol
	Hydroxyethylcellulose

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	Benzoic acid
	Saccharin sodium
	Sodium hydroxide
	Cherry flavoring agent

DEVICE APPLICATION

Play lubricants in variations are used to reduce vaginal dryness and discomfort during sexual se.

INDICATIONS

To reduce vaginal dryness and discomfort during sexual intercourse.

CONTRAINDICATIONS

Sensitivity to any of the components that make up the lubricant.

PRECAUTIONS FOR USE AND POSSIBLE SIDE EFFECTS

is not a contraceptive and it doesn't contain spermicide. The lubricants may slow sperm down, so if you are trying to conceive, talk to your doctor before use. Avoid contact with eyes. Stop using in case of irritation. Talk to your doctor if the irritation persists or if it is required to use the lubricant regularly.

Durex® Play Tingle is not suitable for anal sex.

Description and application.

For bottle with dispenser	For bottle without dispenser
Open by turning the upper part of the package, carefully squeeze some lubricant and apply to intimate areas.	Open the cap, carefully squeeze some lubricant and apply to intimate areas.
The lubricant can also be applied to the outside surface of a condom*. Easily washed off with water. The products are suitable for all types of sex, as indicated on instructions for use. Safe in case of possible swallowing.	

* Durex® Play Tingle is not suitable with NRL condoms

TRANSPORTATION AND STORAGE. SHELF LIFE

Store at the following temperature 10 - 30°C, out of reach of children and away from direct sunlight. An opened package should be used within three months

Transportation is allowed under the following temperature conditions from -10 up to +60°C

ay Feel lube- 3 years;
 ay Tingle lube- 3 years;
 ay Heat lube; – 3 years;
 ay Sweet Strawberry lube- 2 years;
 ay Very Cherry lube- 3 years.

ng on the bottle

bol	Interpretation
	<p>Name: Conformité Européenne — European Conformity</p> <p>Description: indication that the product complies with the requirements laid out in the European Directive</p>
	<p>Name: Batch number</p>
	<p>Name: Use by</p>

Additional information can be printed on the bottle including reference to international standards.

GUIDELINES FOR DESTRUCTION AND DISPOSAL

The devices should be used in accordance with the instructions for use.

Durex Play® lubes are intended for use as a personal lubricant for home use only, and can be disposed of as household waste. No specific requirements are applied for disposal of this product.

MAINTENANCE OPERATION AND REPAIR

This section is not applicable for this range of medical devices

ENVIRONMENT SAFETY REQUIREMENTS

With proper use, storage and disposal, Durex® Play lubricants in variations have no negative impact on the environment.

WARRANTY OBLIGATIONS

Durex® Play lubricants comply with the requirements of the EU Council Directive 93/42/EEC and relevant legislation. Device quality is guaranteed with proper storage and use.

COMPLAINTS

of consumer complaints regarding the device, contact with legal representative in Russia
Reckitt Benckiser Healthcare LLC.

Address: 16/2 Tverskaya st., Moscow, Russia, 125009

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