**INSTRUCTION FOR USE**

**Nozzles for the nasal aspirator "Otrivin Baby"**

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**Manufacturer of the medical device**

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**Intended use of the device.**

Refill nozzles for the nasal aspirators "Otrivin Baby" and "Otrivin Baby Comfort" are used in children (newborn babies, infants and young children) to gently remove bothering nasal secretions.

**Principle of operation.**

**Principle of operation:** Mechanical Aspiration/Suction

**Mode of action:** The compressed air generated by mouth aspiration, passing through a narrow tube, creates a vacuum which sucks the mucus and the mucus gets absorbed in the foam present in the nozzle.

The principle of operation of aspirators is based on the removal of mucous secretions from the nasal passages of the child by blowing into air using uniform openings through the mouthpiece. Refill nozzles, which are inserted into the nasal passage of the child, contain an absorbing filter that traps mucus.

Removal of nasal secretion helps to prevent further complications such as ear infections (otitis), sinus infections (sinusitis) or cough.

Nasal Aspirator and Disposable Nozzle Refills clears your child's nose by gently removing nasal secretions.

The nozzle refills of Nasal Aspirator and Disposable Nozzle Refills (Hard nozzle) are designed to adjust to the child's nose. Each refill includes an absorbent filter to retain mucus, thus the process is completely hygienic.

Nasal Aspirator and Disposable Nozzle Refills (Soft) ergonomic shape and soft touch ensure a better handling; gentle to the baby's nose; perfect and safe adjustment to the baby's nose. Each refill includes an absorbent filter to retain mucus, thus the process is completely hygienic.

Easy and hygienic to use, Nasal Aspirator and Disposable Nozzle Refills (Hard) and Nasal Aspirator and Disposable Nozzle Refills (Soft) helps your baby to breathe, eat, and sleep normally and feel better again.

For hygienic reasons and to avoid contamination, the nozzles are for single use only. Wash your hands carefully before using Otrivin Baby Nasal Aspirator and ensure that the various components are properly assembled. After use, nozzles have to be discarded, the mouthpiece and your hands washed.

#### **Indications and contraindications**

**Indications:** stuffy and congested nose

**Contraindications:**

Do not use the product in case of allergy to any of the device materials.

#### **Warnings**

- Use only the manufacturer's Nasal Refills with the manufacturer's Nasal Aspirator.
- Keep out of sight and reach of children.
- Do not use a broken or damaged product and do not use a used or damaged nozzle or a nozzle with a missing or damaged filter.
- You should consult a doctor before using Nasal Aspirator and Disposable Nozzle Refills following an injury or recent nasal surgery.
- The product should be used on a resting subject. The mouthpiece should be held firmly between the lips.
- For hygienic reasons and to avoid contamination, the nozzles are for single use only.

#### **Precautions**

- Keep the aspirator and nozzles out of the reach and sight of children.
- The aspirator should be used by the parent or caregiver (parents, relatives of the child).

### Refill nozzles for the nasal aspirator "Otrivin Baby"

#### Component parts of a refill disposable nozzle:

- an ergonomic tip allowing to penetrate into the nostril of the child;
- absorbing filter to trap mucus discharge;
- A component of transparent material - allow to control the mucus removal procedure.

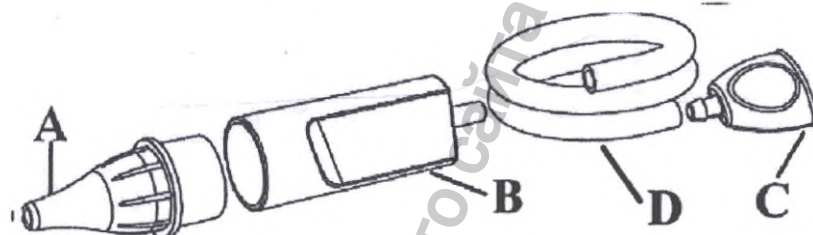
Fig. 1. Diagram of the Otrivin Baby aspirator

#### Device specifications "Nozzles for the nasal aspirator "Otrivin Baby"

Device component	Size, mm
Nozzle inlet diameter (A)	$3.75 \pm 0.17$ mm
Nozzle outlet diameter (A)	$15.55 \pm 0.25$ mm
Nozzle length (A)	$43.77 \pm 0.37$ mm
Filter height	$11 \pm 1$ mm
Filter diameter	$16.00 \pm 0.15$ mm

### Nozzles for the nasal aspirator "Otrivin Baby Comfort"

#### Component parts of a refill disposable nozzle:

- an ergonomic tip allowing to penetrate into the nostril of the child;
- absorbing filter to trap mucus discharge;
- A component of transparent material - allow to control the mucus removal procedure.

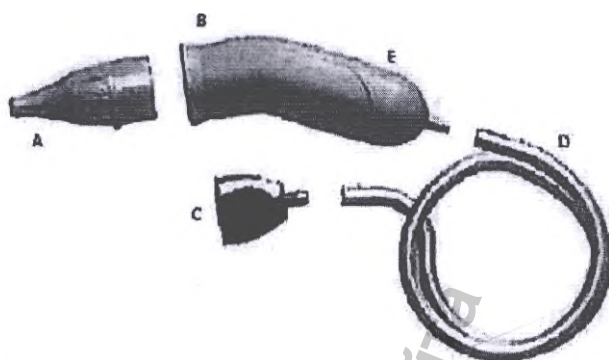
Fig. 2. Diagram of the Otrivin Baby Comfort aspirator.

**Device specifications**  
**“Nozzles for the nasal aspirator “Otrivin Baby Comfort”**

Device component	Size, mm
Nozzle inlet diameter (A)	$3.75 \pm 0,17$ mm
Nozzle outlet diameter (A)	$15.55 \pm 0,25$ mm
Nozzle length (A)	$43,77 \pm 0,37$ mm
Filter height	$11 \pm 1$ mm
Filter diameter	$16.00 \pm 0.15$ mm

**Materials of construction**

Device or part and accessory name (tested)	Materials of construction
<b>Nozzles for the aspirator “Otrivin Baby Comfort”.</b>	
Nozzle	<b>Body:</b> Polyvinyl Chloride AM285AW2178CRY; <b>Filter:</b> Melamine Basotec G+
Bag (package for nozzles)	Polypropylene
Box	Carton Kondor
<b>Nozzles for the aspirator “Otrivin Baby”</b>	
Nozzle	<b>Body:</b> Polypropylene ISPLEN RC 590 S2M; <b>Filter:</b> Melamine Basotec G+
Bag (package for nozzles)	Polypropylene
Box	Carton Kondor

#### **Method of administration:**

1. Before using Nasal Aspirator wash your hands carefully.
2. Fit a soft nozzle (A) into the central part (B) of the device. Do not assemble with a used nozzle.
3. Place the mouthpiece (C) between your lips (parent, caregiver) and insert the end of the nozzle (A) to the opening of the baby's nostril. To improve the performance of the device, it is recommended to hold the other nostril closed during aspiration.
4. Aspire gently, slowly and regularly through the mouthpiece (C).
5. If the nozzle becomes full during an aspiration, change the nozzle, do not re-use.
6. Repeat the operation in the other nostril.
7. Remove and dispose of the used nozzle (A). Wash the nasal aspirator mouthpiece and the central part after use under warm running water and wipe it dry. Wash your hands carefully.
8. Disconnect and discard the tip (A).

#### **Disinfection.**

Wash the nasal aspirator mouthpiece and the central part after use under warm running water and wipe it dry.

Remove and dispose of the used nozzle

#### **Procedure and conditions for utilization/disposal of the medical device**

According to SanPiN 2.1.7.2790-10, utilize this medical device as solid household waste. The device is intended for individual use.

#### **Transportation.**

Transportation is carried out by all types of vehicles in covered vehicles with transportation rules applicable to this type of transport.

Device transportation is carried out at a temperature 0°C to +30°C and humidity up to 65%, non-condensing.

#### **Storage.**

Store the device in its original packaging out of reach of children at a temperature between 0°C and +30°C and humidity of 65%.

#### **Operation.**

Operate the device at a temperature between +15°C and +42°C.

#### **Packaging.**

Nozzles for the nasal aspirator "Otrivin Baby" are supplied in a carton box. Each nozzle is packed in a polyethylene bag. Carton box dimensions: 128.00 ± 0,9 mm x 47.00 ± 0,6 mm x

92.00 ± 0,9 mm. Instruction for use is provided on the carton box. Product weight with packaging 40.7 g ±10%

Nozzles for the nasal aspirator "Otrivin Baby Comfort" are supplied in a carton box. Each nozzle is packed in a polyethylene bag. Carton box dimensions: 128.00 ± 0,9 mm x 47.00 ± 0,6 mm x 92.00 ± 0,9 mm Instruction for use is provided on the plastic box. Product weight with packaging 51.3 g ± 10%

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[www.gosdrazhnadzor.ru](http://www.gosdrazhnadzor.ru)

Package mock-up

Package mock-up for nozzles for the nasal aspirator "Otrivin Baby"

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Package mock-up for nozzles for the nasal aspirator  
 "Otrivin Baby Comfort"

## Labeling

### *Nozzles for the nasal aspirator "Otrivin Baby"*

The package contains the following information:

- Name of the device;
- Composition of the device;
- Manufacturer;
- Manufacturing site;
- Details of the complaint handling organization;
- Marketing Authorization number;
- Storage conditions;
- Shelf life;
- Warnings;
- Instruction for use.
- Lot number
- Best before

### *Nozzles for the nasal aspirator "Otrivin Baby Comfort"*

The package contains the following information:

- Name of the device;
- Composition of the device;
- Manufacturer;
- Manufacturing site;
- Details of the complaint handling organization;
- Marketing Authorization number;
- Storage conditions;
- Warnings;
- Shelf life;
- Instruction for use.
- Lot number
- Best before

## Supply package

### *Nozzles for the nasal aspirator "Otrivin Baby"*

10 nozzle pcs. for the aspirator "Otrivin Baby", packed in a polyethylene bag each, in a carton box. Instruction for use is on a box.

**Nozzles for the nasal aspirator "Otrivin Baby Comfort"**

10 nozzle pcs. for the aspirator "Otrivin Baby Comfort", packed in a polyethylene bag each, in a plastic box. Instruction for use is on a box.

**Warranty and shelf life.**

The manufacturer guarantees the operability of the device throughout the expiration date, subject to the consumer observing the operating rules.

Shelf life for Otrivin Baby Aspirator and refills (Hard nozzle) is 5 years.

Shelf life for Otrivin Baby Aspirator and refills Comfort (Soft Nozzle) is 3 years.

Do not use after expiration date.

**Complaints.**

In the event of complaints about the quality of the medical device or questions related to the use of the medical device, send complaints to the address:

JSC "GlaxoSmithKline Healthcare"

10, Presnenskaya naberezhnaya, Moscow, 123112

Tel. 7 (495) 777 9850

Fax 7 (495) 777 9851

**Toll-free "Hotline" telephone number**

☎ 8 800 333 46 94

FEDELE Françoise  
Global Regulatory Affairs  
Category RA Director, Pain Relief  
09-Jul-2020

GARCIA Pilar  
Global Regulatory Affairs  
Category RA Director, Therapies  
09-Jul-2020

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